Wireless Link Tool
Link Module

OPERATION MANUAL

MADE IN JAPAN

Contents

1. User and Intended Purpose	1
2. Precautions for Handling and Operation	2
3. Package Contents	4
4. Preparation for Use	4
4-1 Installation of the Link Module	4
4-2 Pairing	5
5. Operation Procedure	6
5-1 VarioSurg 3 link operation	6
5-2 Foot control operation	7
5-3 Switching link	8
6. Post-use Maintenance	8
6-1 Preparation for maintenance	8
6-2 Cleaning and disinfection	8
6-3 Periodical maintenance checks	9
7. Spare Parts List	9
8. Specifications	10
8-1 Specifications	10
8-2 Operation principle	10
8-3 Warranty	10
8-4 Disposing product	10
8-5 Symbol	11
9. EMC Information (Electromagnetic Compatibility Information)	12

1 User and Intended Purpose

User: Qualified professional (surgeon with knowledge and skills required for operating this product).

Intended purpose: This product is a link device which enables the Surgic Pro2 foot control to operate the VarioSurg 3. This product is mounted on VarioSurg 3 and communicates via Bluetooth[®] with Surgic Pro2 to enable switching and operation of the two systems (Surgic Pro2 Surgical Micromotor System and VarioSurg 3 Ultrasonic Bone Surgery System) with one foot control.

2 Precautions for Handling and Operation

- Read these safety cautions thoroughly before use and operate the product properly.
- These indicators are to allow you to use the product safely, prevent danger and harm to you and others. These are classified by degree of danger, damage and seriousness. All indicators concern safety, be sure to follow them.

Class	Degree of Risk
⚠ WARNING	Hazard that could result in serious injury or damage to the device if the safety instructions are not correctly followed.
⚠ CAUTION	Hazard that could result in light or moderate injury or damage to the device if the safety instructions are not correctly followed.
NOTICE	General product specification information highlighted to avoid product malfunction and performance reduction.

⚠ WARNING

- TO PREVENT ELECTRIC SHOCK, do not unplug the link cable with wet hands.
- TO PREVENT ELECTRIC SHOCK, be sure to prevent water or antiseptic from getting on the Link Module. If it gets on the Link Module, wipe it off immediately.
- TO PREVENT ELECTRIC SHOCK, use the link cable supplied with the Link Module.
- The system may malfunction when used in the presence of an electromagnetic interference wave. Do not install the system in the vicinity of the device which emits electromagnetic waves. Turn off the main power switch of the VarioSurg 3 if a high frequency oscillation device such as an electrode knife is in the vicinity and used.

⚠ WARNING

- Do not operate close to patients with cardiac pacemakers as there is a danger that it may affect the pacemaker.
- Do not use in rooms with a risk of explosion or near flammable substances. Also, do not use on or near patients under flammable anesthesia (Nitrous Oxide).

⚠ CAUTION

- The Link Module needs special precautions regarding EMC (electromagnetic compatibility) and needs to be installed and put into service in accordance with the EMC information. (See “9 EMC Information (Electromagnetic Compatibility Information)”)
- Portable and mobile RF (Radio Frequency) communications equipment can affect this product. Do not use RF equipment outskirts for the product.
- The use of components other than those specified by the manufacturer may reduce the performance of the Link Module relative to EMC.
- The Link Module should not be used adjacent to, or stacked with, other equipment. If the Link Module must be used adjacent to or stacked with other equipment, make sure that the device operates without malfunction.
- When operating this system, always consider the safety of the patient.
- Do not attempt to disassemble the product nor tamper with the mechanism. Doing so may result in an injury, electric shock or fire.
- Before each use, inspect device operation to make sure there are no abnormalities.
- Especially when using again after a long period of disuse, be sure to inspect before use. If you detect abnormal heat or the like before or during use, stop using immediately and request repair.
- Do not drop, hit, or subject to excessive shock. Doing so could cause malfunctions.
- Make sure each component is securely connected before turning on the power of the VarioSurg 3.
- If there is dirt on the Link Module, remove the link cable and wipe it off thoroughly with a moist cloth. Turn off the VarioSurg 3 before cleaning.

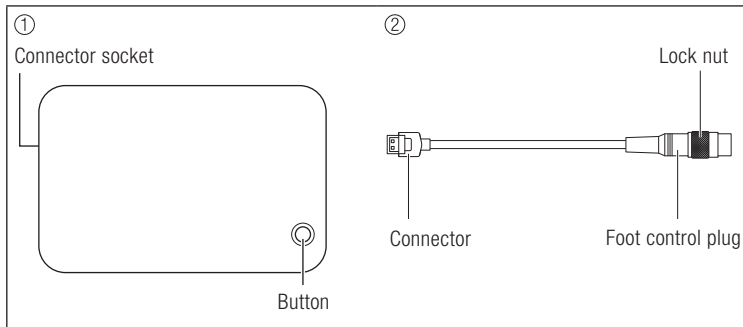
⚠ CAUTION

- The Link Module cannot be sterilized by any method.
- Do not wipe or clean with highly acidic water. Do not immerse in any solutions.
- Do not use this product for purposes other than to link the Surgic Pro2 and VarioSurg 3.
- Read this operation manual and the operation manuals of VarioSurg 3 and Surgic Pro2 carefully before connecting for use. Proper care & maintenance is important for the long term durability of both products. Keep this operation manual within easy reach of users for future reference.

NOTICE

- Turn off the main power switch of the Surgic Pro2 and the VarioSurg 3 control units after each use.
- No special training is required to operate this device.

3 Package Contents


No.	Part name	Quantity
①	Link box	1
②	Link cable	1


4 Preparation for Use

4-1 Installation of the Link Module

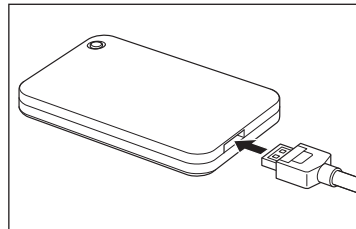① Insert the connector of the link cable into the connector socket of the link box.

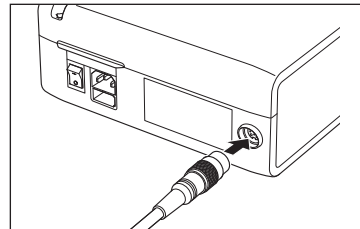② Insert the foot control plug of the link cable into the foot control cord connector socket of the VarioSurg 3 and tighten the lock nut firmly.

⚠ CAUTION

- Install or remove the Link Module only when the power of the VarioSurg 3 is OFF.
- Avoid unnecessary removal of the link cable. Poor contact of the connector can cause malfunction.

4-2 Pairing

To use the Link Module, you must first pair the Link Module and the Surgic Pro2.

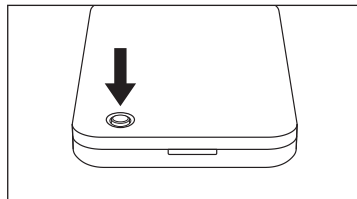
Surgic Pro2 operation


- 1 Turn on the main power switch of Surgic Pro2.
- 2 Press and hold the Setting key for 2 seconds or more to display "BLE SET".
- 3 Press the Enter key.
- 4 Press the TORQUE key and select "LK".
- 5 Press the Enter key.

Link Module operation

- 6 Turn on the VarioSurg 3. The LED of the Link Module blinks white.
- 7 Press and hold the button on the Link Module.

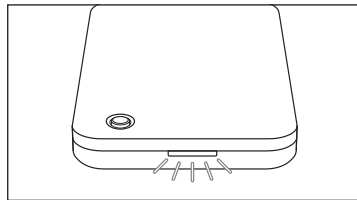- 8 When the LED turns blue, release the button.

- 9 Pairing is successful when the Surgic Pro2 beeps and displays "OK". After pairing, the Surgic Pro2 is automatically connected to the Link Module. During connection, the LED of the Link Module blinks green and the Bluetooth mark blinks on the left of the "VARIOSURG" icon on the Surgic Pro2 display.

- 10 Press the Cancel key of the Surgic Pro2 twice to end setting mode.

⚠ CAUTION

- If any abnormalities are found with the Surgic Pro2 and the VarioSurg 3, refer to the respective operation manuals.
- If the button is not released after the LED turns blue, the LED blinks white and pairing stops. To pair, release the button, and after the LED turns white, proceed from step 7.

NOTICE

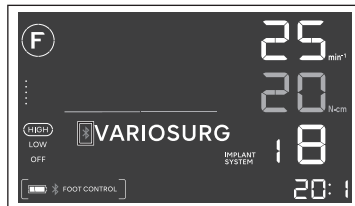
- Pairing operation is required only for the first time. After that, the Link Module is automatically connected when the Surgic Pro2 and the VarioSurg 3 are turned on. To unpair the Surgic Pro2 and the Link Module, the Surgic Pro2 needs to be operated. Refer to the Surgic Pro2 operation manual for details.

5 Operation Procedure

Turn on the paired Surgic Pro2 and the VarioSurg 3 connected to the Link Module.

5-1 VarioSurg 3 link operation


- 1 Make sure that the LED of the Link Module blinks green and the Bluetooth mark is on the left of VARIOSURG icon on the Surgic Pro2 display.
- 2 Press and hold the LINK key of the Surgic Pro2 for more than 2 seconds. The Surgic Pro2 displays "LINK" and the VarioSurg 3 can be operated with the foot control.

⚠ CAUTION

- If external interference is received during connection, the connection can be cancelled.
- The VarioSurg 3 cannot be operated except when the Surgic Pro2 displays "LINK". Check the display of the control unit before treatment.
- Do not turn off one of the control units or unplug the power cord during operating the Link Module. Do not remove the link cable.

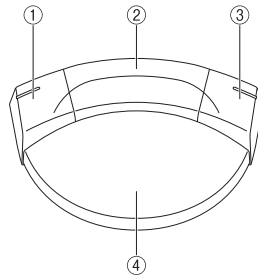5-2 Foot control operation

① Left button

Select coolant solution flow level from 5 levels. If the button is depressed when the flow is set to level 5, it returns to level 1. (Level 0 cannot be selected for the Coolant Flow.)

Press and hold the button for more than 2 seconds: Switching to the linked device. It switches operation to the Surgic Pro2 or the device linked to the Surgic Pro2.

② Center button

Select a program number.

Press and hold the button for more than 2 seconds: Return to the previous program.

③ Right button
Select a Burst Output Level

④ Speed control pedal
Vibrate the tip at the set power while the pedal is depressed.

⚠ CAUTION

- Changing the settings of the foot control buttons of Surgic Pro2 is enabled only when operating Surgic Pro2.

NOTICE

- When the Surgic Pro2 displays "LINK", key operations other than the LINK key and Cancel key are disabled.

5-3 Switching link

Press and hold the LINK key of the Surgic Pro2 for more than 2 seconds to switch the link connection.

Press and hold the Cancel key of Surgic Pro2 for more than 2 seconds to return to the operation of Surgic Pro2.

6 Post-use Maintenance

6-1 Preparation for maintenance

- ① Wear eye protection, a mask, and gloves to prevent infection.
- ② Turn off the power switch of the VarioSurg 3 control unit.

6-2 Cleaning and disinfection

- ① Wipe the surface of the products with a cloth moistened with ethanol for disinfection.
- ② Store the products in a place where they can be kept clean and dry until used next time.

⚠ CAUTION

- Do not clean, immerse, or wipe with electrolyzed-oxidizing water (strong acidic water, super acidic water), strong acidic and strong alkaline liquid solution, solvent containing chlorine, benzene or thinner.
- Follow any additional local directives, standards, and guidelines for cleaning, disinfection and sterilization.
- For maintenance of the Surgic Pro2 and the VarioSurg 3, refer to the operation manual for each product.
- Use ethanol for disinfection that complies with the regulations of each country.

6-3 Periodical maintenance checks

Every 3 months perform periodical maintenance checks, referring to the check sheet below. If any abnormalities are found, contact an authorized NSK dealer.

Points to Check	Details
Link Module	Make sure that the Link Module is securely connected to the VarioSurg 3.
	Make sure Bluetooth communication is executed normally.
	Make sure the LED illuminates normally.

7 Spare Parts List

Model	REF
Link Box	Z1402010
Link Cable	Z1402051

8 Specifications

8-1 Specifications

Model	LMP2
Dimensions	W100 x D55 x H10 mm (Link box)
Weight	67.5 g (with link cable)

Frequency band	2.4GHz ISM band (2.402-2.480GHz)
Transmitting power	Class2 : 2.5mW [dBm]
Modulation	GFSK
Channels	40channel 2MHz spacing

	Temperature	Humidity	Atmospheric Pressure
Use Environment	0 - 40 °C (32 - 104 °F)	30-75% RH	700-1,060 hPa
Transportation and Storage Environment	-10-50°C (14-122°F)	10-85% RH	500-1,060 hPa

No moisture condensation in the Link Module.
Use outside of these limits may cause malfunction.

The Bluetooth® word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. and any use of such marks by NAKANISHI INC. is under license. Other trademarks and trade names are those of their respective owners.

8-2 Operation principle

Power is supplied by attaching to the VarioSurg 3. By pairing with the control unit of Surgic Pro2 via Bluetooth, the VarioSurg 3 can be operated using the foot control of Surgic Pro2.

8-3 Warranty

NSK products are warranted against manufacturing errors and defects in materials. NSK reserves the right to analyze and determine the cause of any problem. Warranty is voided should the product be not used correctly or for the intended purpose or has been tampered with by unqualified personnel or has had non NSK parts installed.

Replacement parts are available for seven years beyond discontinuation of the model. Contact an authorized NSK dealer if repairs are necessary.

8-4 Disposing product

In order to avoid the health risks to operators handling the disposal of medical equipment, as well as the risks of environmental contamination caused thereof, a surgeon or a dentist must to confirm the equipment is sterile.

Ask specialist firms who are licensed to dispose of specially controlled medical wastes to dispose the product for you.

8-5 Symbol

Dispose of this device and its accessories via methods approved for electronic device and in compliance with the Directive 2012/19/EU.

See operation manual

Marking on the outside of equipment or equipment parts that include RF transmitters or that apply RF electromagnetic energy for diagnosis or treatment.

The EU directive 93/42/EEC was applied in the design and production of this medical device.

Manufacturer

Authorized representative in the European Community

Keep dry

Handle with care

This is the correct upright position of the distribution packages for transport and/or storage.

SN

Serial No.

REF

Catalog number

Temperature limitation

Humidity limitation

Atmospheric pressure limitation

Caution, Consult operation instructions.

9 EMC Information (Electromagnetic Compatibility Information)

Refer to the operation manual of Surgic Pro2 (OM-E0858E).

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NSK Europe GmbH